PRÉLÈVEMENT

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BLOOD COLLECTION

Go to YouTube by
scanning the Following
QR code to view the
demonstration video.

EN

INTRODUCTION

COVID-19 is an acute respiratory illness caused by infection with SARS-CoV-2. SARS-CoV-2 belongs to the same family of viruses that cause Severe Acute Respiratory Syndrome (SARS). The most common symptoms of COVID-19 are fever, fatigue and a dry cough. Some patients may experience pain, nasal congestion, runny nose, sore throat, diarrhea, and loss of taste and smell. These symptoms are usually mild and begin gradually. Some people are infected but do not develop any symptoms. Most people (about 80 %) recover from the disease without the need for hospital care.

The disease can be spread from person to person through small droplets emitted from the nose or mouth when a person infected with COVID-19 coughs or exhales. Transmission through contact of contaminated hands with the face is common. Hands can become contaminated after touching surfaces that carry the virus. The estimated incubation period for COVID-19 ranges from 1 to 14 days.

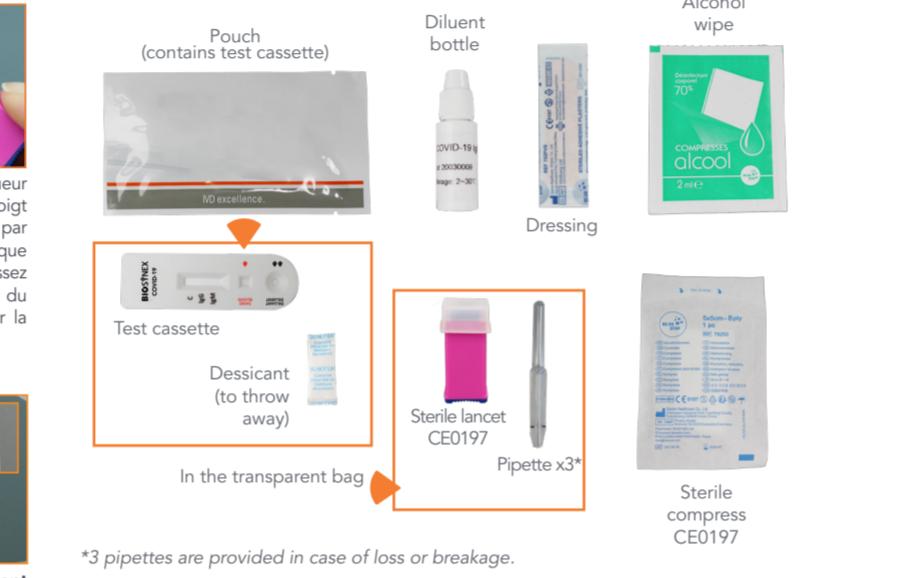
INTENDED USE

COVID-19 self-test is a rapid immunochromatographic self-test for the qualitative detection of IgG and IgM antibodies against SARS-CoV-2 from a drop of capillary blood. The test comprises SARS-CoV-2 antigens (called RBD - Receptor Binding Domain) coupled with colloidal gold particles that are capable of detecting IgG and IgM antibodies directed against SARS-CoV-2. This self-test determines whether you have been in contact with the virus responsible for COVID-19. The COVID-19 self-test, based on antibody detection, can identify individuals who are unaware that they have been infected, either because they have never developed symptoms or because they have symptoms that have never been properly diagnosed. This means that self-testing can identify silent infections, as well as people who have been sick but have recovered. This test is for use only in adults or under adult supervision if performed on a child.

The COVID-19 self-test will give a positive result in the majority of cases after 13 days of the onset of symptoms. A negative result with COVID-19 Test in a symptomatic individual does not exclude COVID-19 infection. If symptoms suggestive of COVID-19 are present, a negative result should be followed up with a second test 1-2 days after the first test.

This self-test does not detect the presence of the virus responsible for COVID-19, but it does detect the response of a patient's immune system against the COVID-19 virus.

KIT CONTENTS

*3 pipettes are provided in case of loss or breakage.

Ref 859081 contains one bag for one self-test.

TESTING GUIDELINES

• Wash your hands with warm water beforehand and dry them properly. Warm water improves circulation at the fingertips.

• You can let your hand hang for about a minute along your body so that more blood can flow into it. It is recommended that you prick your left hand for a right-handed person and your right hand for a left-handed person.

• It is best to prick the fingertip on the side: this is where there is the most blood and less sensitivity. Avoid areas where the skin is thicker and therefore more difficult to prick.

• It may happen that a drop does not form right away. In this case, you should wait a while and massage your finger lightly, from the base to the puncture site.

• Do not hesitate to form large drop of blood at the tip of your finger. This way, the pipette will fill up with blood easily and quickly.

LIMITATION OF THE TEST

• This test is not a PCR screening test to detect the presence of the virus, but it does detect a reaction of the patient's immune system against COVID-19.

• This test can be used to identify silent infections with no symptoms, past or present, and people who have been sick but have recovered.

• This test does not determine whether the patient is contagious, it only indicates whether the person has been in contact with the virus at some point.

• A false negative result may occur in some patients with early symptoms. Repeat the test on subsequent days to detect the development of antibodies. At the onset of infection, anti-SARS-CoV-2 IgM concentrations may be below detectable levels.

• The test will only indicate the presence of anti-SARS-CoV-2 antibodies in the sample and should not be used as the sole diagnostic criteria.

• This test is not a substitute for a medical consultation or for the result of a biological analysis performed in a medical laboratory.

• Incorrect results may be obtained:

- If the test is not used according to the instructions in the package insert,

- If the foil pouch is damaged or if the test is not performed immediately after opening the foil pouch,

- If the storage conditions are not met or if the test is performed after the expiry date on the foil pouch.

• A positive result must be confirmed by laboratory analysis. Consult your doctor.

• There is a possibility of obtaining false positive results, especially in some cases of patients with rheumatoid arthritis

• Cases of cross-reactions have been observed with SARS-CoV-1 and other seasonal coronaviruses.

• If you have a positive result showing the presence of antibodies, it does not mean that you are immune to COVID-19.

• The response of the immune system is unique to each individual. Antibodies may disappear in some individuals as soon as a few weeks after infection.

Refer to the interpretation section to read the result.

PERFORMANCES

Sensitivity and specificity

Biosynex COVID-19 self-test was compared to the clinical diagnosis (confirmed). The study involved 456 samples for IgG and 466 samples for IgM.

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A feasibility study demonstrated that:

- 94,5 % of the different types of results were interpreted correctly,

- 88 % of non-professionals carried out the test without requiring assistance.

INTERFERENCES

None of the following substances at the tested concentration showed any interference with the test.

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