



DISINFECTANT FORMULATION

WPH202S

Class IIA Medical Device registered with the Italian Ministry of Health
Registration ID BD/RDM 1898970

What is

Disinfectant formulation of non-invasive medical devices surfaces and confined environments surfaces such as equipment, operating tables and medical units. Professional use only. Ready-to-use bactericidal, sporicidal, fungicidal and virucidal product.

Application

The product is dispensed in the measure of 1ml/1m³ of volume to be treated using a device that generates a specific aerosolization. The programming of the above mentioned device is done through the acquisition of the information stored in the microchip previously prepared in the environment to be treated, identifying it in a unique way. The time to cover an environment of 100 m³ is about 4 minutes.

Effectiveness: the effectiveness of the disinfection activity has been demonstrated in compliance with:

MEDICAL AREA

EN 13727:2015

Chemical Disinfectants and Antiseptics - Quantitative suspension test for the evaluation of bactericidal activity in the medical area

EN 14347:2005

Chemical disinfectants and antiseptics - Basic sporicidal activity

EN 14348:2005

Chemical disinfectants and antiseptics - Quantitative suspension test for the evaluation of mycobactericidal activity of chemical disinfectants in the medical area including instrument disinfectants

EN 14476:2013

Chemical disinfectants and antiseptics - Quantitative suspension test for the evaluation of virucidal activity in the medical area

EN 13624:2013

Chemical disinfectants and antiseptics - Quantitative suspension test for the evaluation of fungicidal or yeasticidal activity in the medical area

EN 14561:2006

Chemical disinfectants and antiseptics - Quantitative carrier test for the evaluation of bactericidal activity for instruments used in the medical area

AFNOR NF T 72 190:1988

Water-miscible contact disinfectants used in liquid state Determination of the mycobactericidal and sporicidal activity for surface decontamination

Corporate Headquarters:
Via F. Ozanam, 4 • 20129 Milano (MI) • T. +39 02 36705859 • F. +39 02 36705880
Operational Office:
Via Rossino, 5 • 20871 Vimercate (MB) • T. +39 039 6080590 • F. +39 039 6388615
Web: www.microdefender.it • VAT No. 09792140965

